

Diapouch™ - Instructions for Use.

WATERPROOF POUCH TO HOLD AND PRESERVE CATHETER TUBES

Sterile, Elastic, Soft Waterproof Pouch

Holds and preserves catheter tubes and connectors of Hemodialysis/Peritoneal/Oncology catheter

Protects and manages entry/exit sites of the catheter and its tubing and connectors.

During showering, provides fair protection against water penetration into the pouch and around the exit site*

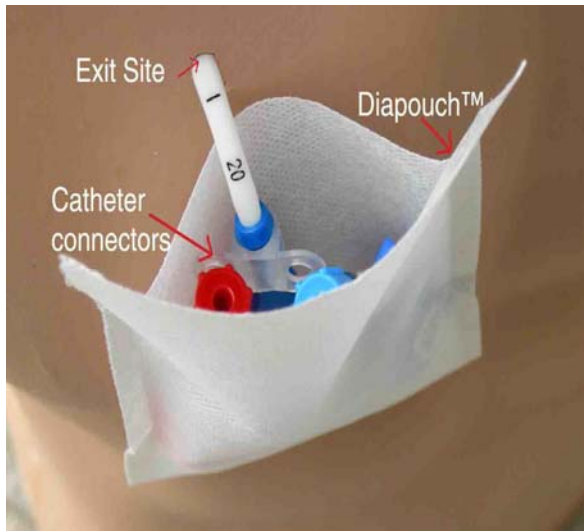



Fig. I



Fig. II

Method of Use:

1. Peel the pack open at its edges
2. Remove the adhesive tape from the pouch
3. Remove the silicon paper from the rear adhesive strip, or leave it and place the catheter connectors in the pouch (**Fig I**) keeping the shorter edge facing the skin (it is recommended to cover the connectors with a 10 x 10cm gauze sponge before placing them in the pouch).
4. Fit a slit sponge around the exit site
5. The longer flap of the pouch should cover the slit sponge completely
6. Ensure that the skin is completely dry before adhesion of adhesive tape.
7. Peel off the silicon paper from the adhesive tape, and adhere the tape around the upper edge of the sachet, leaving approximately 2 cm margins from each edge of the upper flap. (**Fig II**). This procedure will also cover the pouch area containing the catheter ends.
8. Ensure that the adhesive tape is securely fixed around the pouch to prevent water penetration on all sides, especially over the exit site area.*.

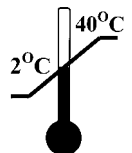

- DIAPOUCH™ - Waterproof Pouch outer size: 17 X 8 cm, Inner size: 13 X 7cm / Or: 19 X 8 cm, inner size 15 X 7cm
- ADHESIVE PLASTER air permeable, size:10X12 cm

***Note:** *The pouch is completely water resistant. However, pouch flap should completely cover the absorbent dressing around the exit site, with no extruding edges. The adhesive tape should be applied flatly with no kinks and cover the entire flap. Ensure that the skin is completely dry before adhesion!*

Pollak Ltd. Hamlacha 20, Rosh Ha'ayin, Afek, 48091 Israel.

Tel: 972 3 9022885 Fax: 972 3 9024531 euroband@hotmail.com

www.euroband.com

EC REP L.C.H. 7 rue de Thionville
75019 Paris France

EUROBAND

REF: 01143 - 01148



STERILE EO

CE 0473

