

SilvalGuard™

Cat. # 01147/4

Protects Exit Site of Extruding Orthopedic Pin

External fixation devices are widely used by medical personnel when treating patients with severely broken limbs.

Infections frequently develop at the Exit Site of these pins.

SilvalGuard™ protects the Exit Site from becoming infected.

The dressing can remain in place for up to 7 days!

Fits easily around each pin; does not interfere with connecting rods.

Patient can shower regularly wearing SilvalGuard.

Pack Contains 4 sets of:

Silvalpad™ († slit pad containing releasable **silver** ions).

Two U-shaped porous adhesive tapes.




Worldwide Patent Pending

Method of use:

Peel apart the sterile pouch. Position **Silvalpad** (2) around the splint (1) (with brown surface facing the skin), secure **Silvalpad** (2) in place by means of the "U" plaster adhesive strip (3) and (4) facing one another with the "U" shape fitting around the splint.

In order to activate the silver ions, the Silvalpad pad must be wet. After the bandage is completely fixed around the pin – inject (about 3ml) water onto the pad. Water will be absorbed into the pad. Repeat this procedure whenever you feel the pad is dry.

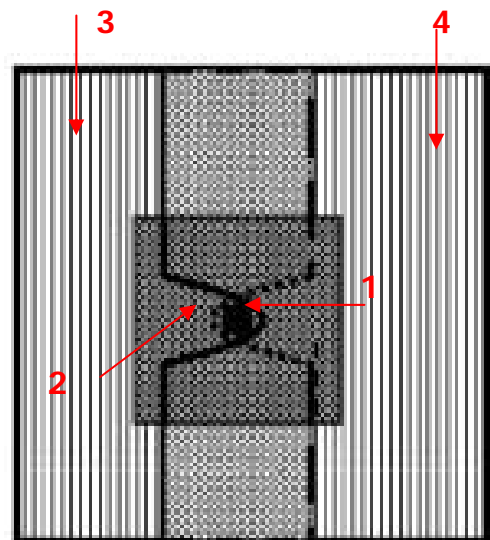

Pollak Ltd. Hamelacha 20, Rosh Ha'ayin, Afek, 48091 Israel.

Tel: 972 3 9022885

Fax: 972 3 9024531

euroband@hotmail.com

www.euroband.com

EUROBAND

